Rolyan® Dynamic Wrist Splint:

A352-200, 202, and 204

The figures referred to in the instructions are on page 6.

INDICATIONS

- For stiff wrists, soft wrist contractures, some wrist fractures, crush injuries and following nerve and tendon repair

CONTRAINDICATIONS

- Contraindicated in the presence of unstable wrist fractures
- Contraindicated in the presence of fixed contractures

PRECAUTIONS

- This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- Adjustments should be made only by a healthcare professional.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- The splint should fit comfortably without interfering with circulation.
- After heating splinting material, always check its temperature before applying it to the patient.
- Check all cap screws and set screws to be sure they are tight before applying the splint to the patient.


CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water. The straps can be washed

with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication.

INSTRUCTIONS FOR USE

Splint components are referred to by the following names:

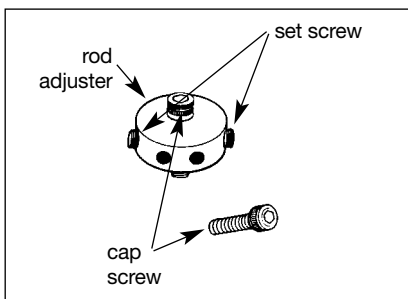1. The splint will fit either hand. The cutout portion of the palmar cuff should be between the thumb and first digit (A and B). When inverting the palmar cuff, follow these steps:
 - Remove the end protectors from the long wires.
 - Loosen the set screw on the rod adjuster on each side of the palmar cuff as shown in C and remove the cuff from the long wires.
 - Invert the cuff and replace it, tightening the set screws to hold it in place, and replace the end protectors.

Modifying the splint

2. The splint, which is made from 1/8" (3.2mm) Polyform® splinting material, can be modified slightly for a better fit. Spot heat it with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C) for one minute or until pliable. Avoid heating the metal splint attachments, and do not heat the splint near any of the screws. **Heating the entire splint**

may cause it to lose its preformed shape.

Polyform has a high degree of drape—for better results when modifying the material, shape it with stroking movements.

3. The distal/proximal position of the palmar cuff can be adjusted. Loosen the set screw on the rod adjuster on each side of the palmar cuff as shown in C, adjust the position of the cuff on the long wires, and tighten the set screws.
4. The hinges can be adjusted laterally. Using the larger allen key, loosen the four cap screws on the distal edge of the forearm splint surface, move the hinges farther apart or closer together, and fasten the screws securely.
5. The distal/proximal position of the hinges can be adjusted. Loosen the two proximal cap screws that fasten the hinges to the forearm splint portion, move the metal attachment to position the hinges appropriately, and tighten the cap screws.

Applying the splint

6. To increase extension, apply the forearm portion of the splint to the dorsal surface of the forearm as shown in A.
7. To increase flexion, invert the palmar cuff (see instruction 1) and apply the forearm portion of the splint to the volar surface of the forearm as shown in B.
8. With the splint in position on the patient, fasten the two large straps to the forearm splint section, one distally and one proximally.

Adjusting the amount of traction force

9. Loosen the cap screw in the center of the rod adjuster on both sides as shown in D. If necessary, use the wrench to hold the nut on the inside of the hinge.
10. Rotate the long wires to increase or decrease the amount of traction force, and tighten the cap screw in the center of each rod adjuster.

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Patterson Medical Ltd.