## Rolyan® Dorsal Blocking Splint:


A353-1, 2, 1E, 2E, 270, and 370; 5514-32 through 5514-35; 5504-36 through 5504-37; 5504-41 through 5504-42

A

B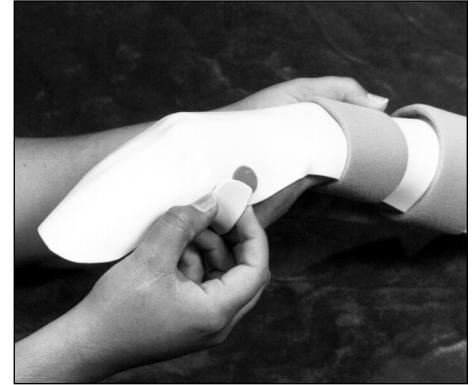

C

### INDICATIONS

- For immobilizing the wrist and MCP and IP joints of the fingers
- May be used following flexor tendon repair and a passive or active tendon implant

### CONTRAINDICATIONS

- Not in the presence of severe edema
- Not in the presence of developing flexion contractures
- Not for use directly over open wounds and infected areas

### PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**

- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

### CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

### PROCEDURE

The splint blank can be applied to either hand—simply invert the splint blank.

1. Heat splint blank in water heated to approximately 160°F (70°C) until pliable. (Refer to the instructions for Rolyan splinting materials for information about working with specific splinting materials.)

2. Remove from heat pan and place on a towel on a flat surface. Blot excess water.
3. Standing in front of the patient, position the splint with the palmar bar on the ulnar side of the hand. Mold splint over the dorsum of the forearm and then the digits.
4. Mold the palmar bar around the ulnar side of the hand (A) and through the thumb web space (B). Note that, if the splinting material has not been coated to prevent accidental bonding, the palmar bar should not be allowed to touch the splint; material that has been coated can be popped apart when cool.
5. The splint can be trimmed and the edges flared if necessary.
6. Apply appropriate straps. Hook and loop material can be fastened to the splint to secure the palmar bar. A completed splint is shown in C.

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**Patterson Medical Ltd.**