## Rolyan® Economy Stabilized Knee Sleeve with Removable Buttress:

A403-810 through A403-814

## Rolyan® Lateral J Support:

A403-820 through A403-829

### INDICATIONS

- To help reduce pain and provide positioning for patellofemoral disorders

### CONTRAINDICATIONS

- Not for use with severe edema
- Not for people who have an allergic reaction to neoprene or who have a tendency to develop dermatitis
- Not for use over open wounds and infected areas
- May not be for people with circulatory problems

### PRECAUTIONS

- **The brace should fit snugly and comfortably without interfering with circulation.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**

### CARE AND CLEANING

- Fasten closures. Hand wash fabric in cold water and mild soap. Rinse thoroughly. Allow to air dry at room temperature. Be sure brace is completely dry before replacing hinges and reapplying brace.

- Do not use ointments or oils under the brace.
- Do not dry clean or expose to heat or sunlight for extended periods of time.

### INSTRUCTIONS FOR USE

#### Rolyan Economy Stabilized Knee Sleeve with Removable Buttress

1. With wider end up and straps open, pull support into position over knee.
2. Center cut-out area over patella and center stays along joint line.
3. Starting with lower strap, feed straps through D-rings and secure them.
4. Buttress can be repositioned to provide appropriate support. It can also be trimmed.
5. A properly applied Economy Stabilized Knee Sleeve with Removable Buttress is shown in **A**.

#### Rolyan Lateral J Support

1. With wider end up and strap open, pull support into position over knee.
2. Center cut-out area over patella and center stays along joint line.
3. Fasten strap for positioning and support.
4. A properly applied Lateral J Support is shown in **B**.

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**Patterson Medical, Ltd.**
Patterson Medical Products, Inc.