
Rolyan® Tension-Adjustable Finger Loops:


A444-10 and A444-11

Rolyan® Tension-Adjustable Finger Slings:

A444-13 and A444-14

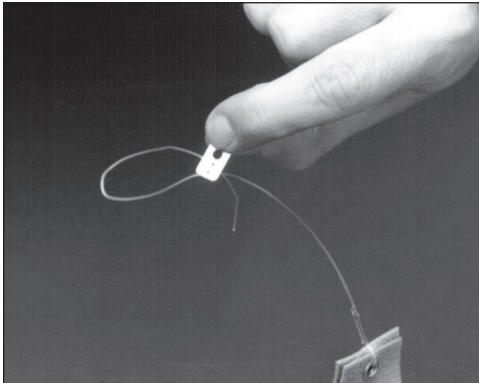A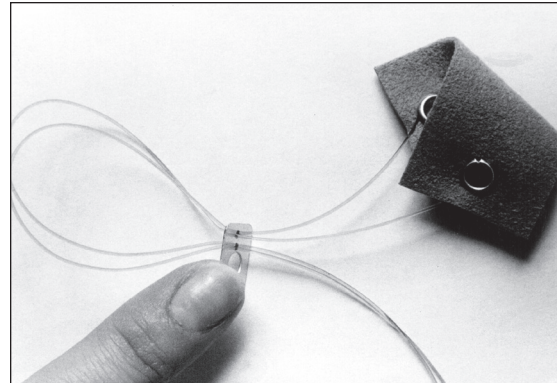

B

Be sure to note which product you are using and follow the instructions that apply to that product. Unless otherwise stated, these instructions apply to both of the products.

Note that finger loops have a single piece of monofilament holding eyelets together (A), while finger slings have two pieces of monofilament, one attached to each eyelet (B).

INDICATIONS

- For use with a traction device for low-profile dynamic and static progressive splinting

CONTRAINDICATIONS

Finger loops

- Not for use with severe edema

PRECAUTIONS

- *This product is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The*

healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.

- *If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.*

Finger slings

- *Be sure the eyelets clear the finger to avoid possible irritation.*

CARE AND CLEANING

Hand wash with warm water and mild soap. Allow to air dry thoroughly before reapplication.

PROCEDURE

Finger slings

1. To keep lengths of monofilament even while making adjustments, fold sling in half and match up eyelets. Treat the two monofilaments as one.

Finger loops and slings

2. To increase amount of traction force or reduce length of monofilament, feed portion of monofilament that is connected to finger loop or sling up through first small hole in aluminum connecting piece. Pull slack down through second small hole (middle hole).
3. To reduce amount of traction force or increase length of monofilament, feed portion of monofilament that is not connected to finger loop up through second small hole (middle hole). Pull slack down through first small hole.
4. Attach spring or rubber band through large hole in aluminum connecting piece.

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Patterson Medical, Ltd.