



Stacking Cones

Item Sku# 1498

Item Description:

Washable, non-toxic and impact resistant acrylic cones are 4-1/4" high with a 2-1/2" base diameter. Six translucent colors: blue, green, red, amber, smoke and clear. Optional bases: White molded ABS plastic base measures 14-1/2" x 8". Wooden base is 14-1/2"L x 7-7/8"W x 3/4"H. Latex free.

Indications:

Impact-resistant Stacking Cones are ideal for upper extremity activities, reaching and grasping, eye-hand coordination, color identification and perceptual training.

Care and Cleaning:

Stacking Cones and Base can be wiped clean with a mild detergent – no bleach, no heavy duty cleaning products - to clean. Dry with soft cloth.

Procedure:

Modify the traditional cone stacking task for your stroke patient:

- Place a stack of cones in front of your patient slightly towards the affected side.
- Instruct your patient to place the affected hand onto the top cone (the patient may use their unaffected hand to grasp her affected wrist to assist with the task if their affected hand is still significantly weak)
- Tell your patient to move the top cone towards the unaffected side to start a new stack of cones, moving one cone at a time.

Accessory Parts Available:

Item Number: 149801

Item Description: 12 replacement cones (two of each color) without base



Distributed by:

Patterson Medical
28100 Torch Parkway Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Direct Phone: 630-393-6000
Fax: 800-547-4333
Direct Fax: 630-393-7600

Patterson Medical
W68 N158 Evergreen Blvd
Cedarburg, WI 53012, USA
Phone: 800-323-5547
Fax: 800-547-4333

Patterson Medical Canada
6675 Mill Creek Drive, Unit 3
Mississauga, ON L5N 5M4
Phone: 800-665-9200
Direct Phone: 905-858-6000
Direct Fax: 905-858-6001

Authorized European Representative
Patterson Medical, LTD.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784