


Universal Medical Products®  
**Personal Sentry Fall Monitoring System**  
**User Instructions**  
(Model 91230)


The Universal Medical Products® Personal Sentry monitor is designed to sound an alarm when a patient either exceeds his or her safe range of movement from a wheelchair, chair, or bed or wishes to call for assistance.

## Monitor Features

---

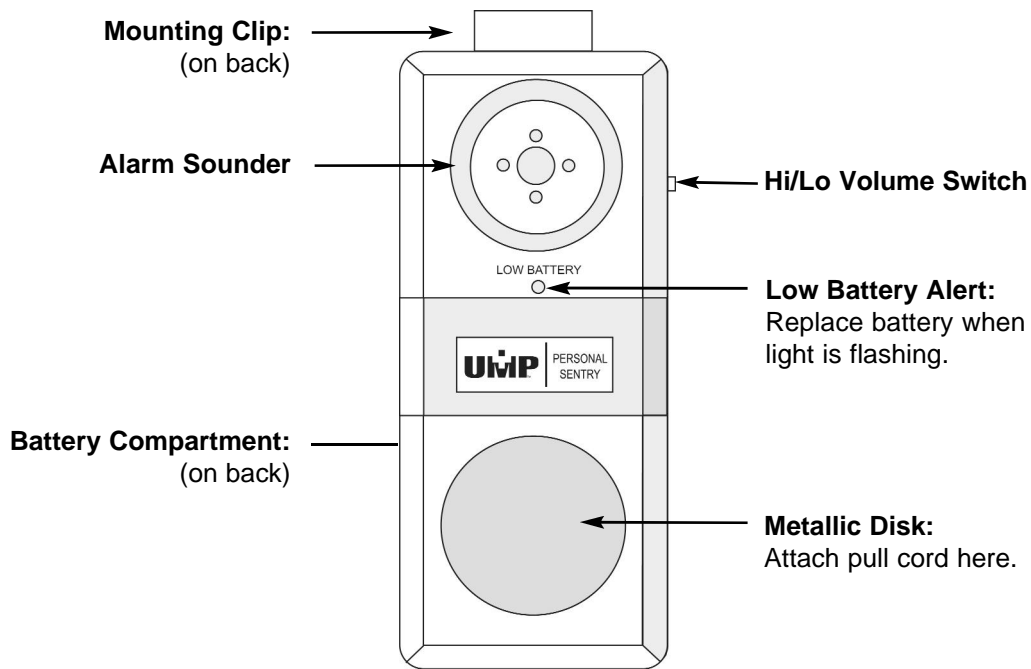

**Cord Length Adjuster**

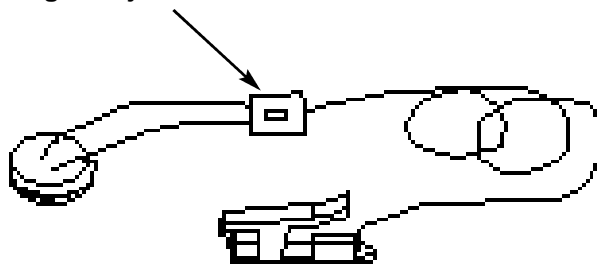

**Adjustable Cord with Locking Clip**

## Installing the Battery

---

Place thumb on the arrow engraved on the battery compartment lid on the back of the monitor and press down firmly in the direction of the arrow to slide the lid off. Insert a fresh, new 9V alkaline battery into the battery compartment making sure to observe proper battery polarity (positive to positive, negative to negative). When the battery is properly installed, slide the lid back into place.

## How to Use the Personal Sentry Monitoring System

---

The monitor is mounted to either a chair or bed using the clip on the back of the monitor or using the optional Universal Mounting Bracket or Chair Strap. Place the magnetic disk from the pull string to the metallic disk on the front of the monitor and attach the locking clip to the resident's garment at the top of the shoulder or back of the neck. The length of the cord needs to be determined by how far you want the monitored person to be able to move before activating the alarm. Use the Cord Length Adjuster to adjust the length of the cord. The clip has a locking tab which must be folded down to open the clip. When the clip is attached, the tab should be flipped up to prevent easy removal by the patient. When the resident moves beyond the length of the cord, the disk pulls from the monitor and the alarm sounds.

Adjust the volume level by using the **Volume Switch** on the side of the monitor. Select either a **Hi** or **Lo** volume.

For added protection for the monitor, an optional cushion protector may be purchased.

## To Check Battery Level

---

The red light on the front is marked LOW BATTERY. To check the battery level, detach the magnet from the face of the monitor. You will hear the alarm sound. If the battery is low, the LOW BATTERY light will flash. Replace the battery immediately.

## **▲ WARNING**

Read and follow these Installation Instructions, particularly the Warnings and Cautions, before using your UMP® fall monitoring system. Failure to do so may result in injury or death.

Test your UMP® fall prevention monitor for proper operation before each use. Check for low battery by removing the magnet from the monitor to activate the alarm. Replace the battery immediately with a new 9V alkaline battery if the LOW BATTERY light is flashing.

Use your monitor in conjunction with your facility's fall management program. UMP® monitors are not a substitute for proper nursing care. The effectiveness of the UMP® fall prevention monitoring system relies entirely on an immediate response by the caregiver to the UMP® monitor alarm.

Keep the UMP® Personal Sentry cord free of all encumbrances at all times. Place the cord on top rather than under pillows. Do not place items on the bed, chair or monitoring location that may interfere with the cord assembly.

Use the proper length of UMP® cord. Using a length outside the caregiver's predetermined safety zone may result in injury to a patient.

Only use an UMP® Personal Sentry cord with the UMP® Safety Sentry monitor. Any other cord, particularly a cord with higher elasticity, may be unsafe and result in injury to a patient.

The UMP® Personal Sentry monitor will not stop a person from leaving a bed, chair, wheelchair or room. It is intended only to alert a caregiver that a person may need assistance. Other interventions may still be required for some people.

The UMP® Personal Sentry monitor can be defeated by a cognitively aware person, a person with only a few moments of lucidity or an uncooperative person. Properly assess each individual before the UMP® Personal Sentry monitor is used.

## **▲ CAUTION**

Do not open the UMP® Personal Sentry case for any reason, or remove the screws that hold the case together. The case is sealed and if the seal is broken, the unit may be damaged and the warranty will be voided. Call UMP® Customer Service if your UMP® Personal Sentry unit requires service.

### **Universal Medical Products® Standard Limited Warranty**

LIMITED WARRANTY. Universal Medical Products® (UMP) products are warranted to be free of defects in materials and workmanship for a period of one year from date of purchase. If any defects covered by this Warranty appear, Stanley Security Solutions, Inc. ("ST") will, at its option, repair or replace the defective component on an exchange basis with new or rebuilt parts at its expense, without charge for parts or labor upon delivery of such product to ST at Customer's expense. EXCEPTION: UMP® PADS ARE WARRANTED TO BE FREE OF MANUFACTURING DEFECTS FOR VARYING PERIODS AS SPECIFIED TO CUSTOMER AT TIME OF SALE. ST is not responsible for warranty service should the UMP® product fail to be properly maintained or fail to function properly as a result of misuse, abuse, neglect, or damage caused by disasters, such as fire or lightning, damage caused by water-immersion, faulty or leaking batteries not supplied by ST, service other than by ST or units in use in violation of instructions furnished by ST. Postage, insurance, or shipping costs incurred in presenting UMP® products for warranty service are Customer's responsibility.

THE FOREGOING WARRANTIES ARE EXCLUSIVE AND IN SUBSTITUTION FOR, AND IN SUBMISSION FOR AND BUYER HEREBY WAIVES, RELEASES AND RENOUNCES, ALL OTHER WARRANTIES, OBLIGATIONS AND LIABILITIES OF ST AND ALL OTHER RIGHTS, CLAIMS AND REMEDIES OF BUYER AGAINST ST, EXPRESS OR IMPLIED, ARISING BY LAW OR OTHERWISE, WITH RESPECT TO ANY NONCONFORMANCE OR DEFECT IN ITEMS OR SERVICES PURCHASED HEREUNDER, INCLUDING, BUT NOT LIMITED TO: (A) ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; (B) ANY IMPLIED WARRANTY ARISING FROM COURSE OF PERFORMANCE, COURSE OF DEALING OR USAGE OF TRADE; (C) ANY OBLIGATION, LIABILITY, RIGHT, CLAIM OR REMEDY IN TORT, WHETHER OR NOT ARISING OUT OF THE NEGLIGENCE OF STANLEY SECURITIES SOLUTIONS, INC. (WHETHER ACTIVE, PASSIVE OR IMPUTED); AND (D) ANY OBLIGATION, LIABILITY, RIGHT, CLAIM OR REMEDY FOR LOSS OF, OR DAMAGE TO, ANY EQUIPMENT.

ST SHALL HAVE NO OBLIGATION OR LIABILITY, WHETHER ARISING IN CONTRACT, TORT OR OTHERWISE, FOR LOSS OF USE, REVENUE OR PROFIT, COST OF CAPITAL, COST OF SUBSTITUTE EQUIPMENT, ADDITIONAL COSTS INCURRED BY BUYER, BUYER'S CUSTOMERS OR OTHER THIRD-PARTIES, FOR DAMAGES RESULTING FROM PERSONAL INJURY OR PROPERTY DAMAGE OR ANY OTHER INCIDENTAL OR CONSEQUENTIAL DAMAGES WITH RESPECT TO ANY NONCONFORMANCE OR DEFECT IN ANY ITEMS PROVIDED HEREUNDER.

Universal Medical Products  
1620 N. 20th Circle; Lincoln, NE 68503-1103  
Tel.: 1-888-716-0077; Fax: (402) 475-4023  
u-m-p.com

**To place an order, contact your distributor**