Rolyan® Static Progressive Finger Flexion Splint:

A542-29 through 34

INDICATIONS

- For fingers with limited DIP and PIP flexion following soft tissue injury or trauma

CONTRAINDICATIONS

- Not for joint contractures that are the result of bony blocks
- Not for unstable fractures of the digit
- Not for use with severe fluctuations in edema in the digit

PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **Be sure that the acorn nut is tight on the end of the screw.**

CARE AND CLEANING


- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot

- water, open flames, and sunny car windows.
- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.
- The Plastazote® liner may be wiped with cool water and mild soap.

INSTRUCTIONS FOR USE

The splint consists of:

- hinged splint made from 1/16" (1.6mm) Aquaplast®-T splinting material with 1/16" (1.6mm) Plastazote liner, 2 D-ring straps and 2 long adjustment bolts
 - 1 additional optional self-adhesive D-ring strap
1. Put the splint into position on the dorsum of the finger. The wider, flanged area fits proximal to the MCP and the narrower, tapered end fits around the finger nail area. Position the cut-out areas over the axes of the DIP and PIP joints.
 2. Fasten the proximal and distal straps around the finger. If desired, add the third strap.
 3. Check for fit as follows:
 - Be sure the straps clear the joint creases. If necessary, trim the width of the straps.

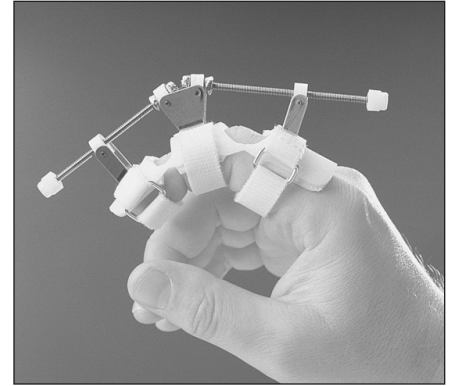A

- If the splint is too long or the sides extend past the circumferential mid-point of the finger, the material can be trimmed with a scissors. **Do not heat the splint for trimming.** Avoid trimming the sides into the cut-out joint areas.
4. By turning the bolts, set the splint to provide flexion near the available end range of motion.
 5. As the condition improves, the amount of flexion provided can be adjusted by turning the bolts.
 6. A properly applied splint is shown in A.

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Patterson Medical, Ltd.