Rolyan® Finger Gutter Splint: A580-1 through A580-5 and A580-7

INDICATIONS

- To immobilize PIP and DIP joints
- May be used following fractures, dislocations, ligamentous injuries, and fingertip injuries

CONTRAINDICATIONS

- Not for use with severely fluctuating edema
- Not for use directly over open wounds and infected areas

PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**


- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water. It should be allowed to dry thoroughly before reapplication.

APPLICATION

1. Splint can be widened without heating by gently bending edges outward. If needed, splint, which is made from 1/16" (1.6mm) **Ezeform®** Light splinting material, can be spot-heated with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C). **Heating the entire splint may cause it to lose its preformed shape.**

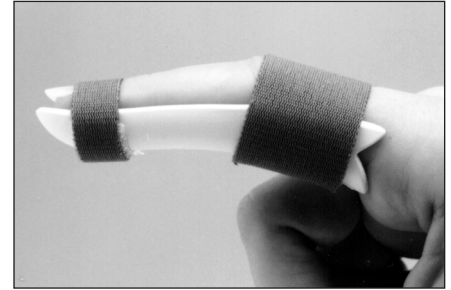A

2. With splint on finger, attach straps as shown in **A**, or use tape or a cohesive bandage to hold splint in place.

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© June 1992, Revised 1/98, 1/02, 04/03, 6/03, 11/07
Patterson Medical Products, Inc.