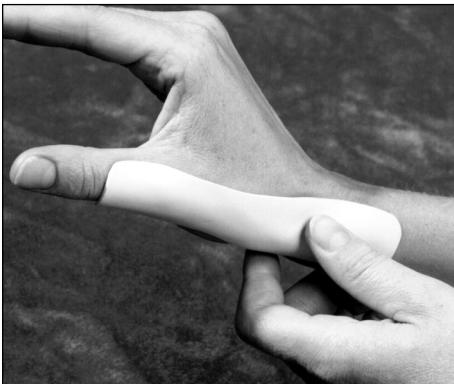
Rolyan® Gel Shell® Thumb Spica Splint:

A599-001 through 008 and 010

A

B

C

INDICATIONS

- To help protect the thumb to allow healing following injury
- May be used for conservative treatment for deQuervain's syndrome and to decrease pain and hypertrophic scarring following surgery and suture removal

CONTRAINDICATIONS

- Not for severe sprains, immediate treatment following fractures, or if rigid support is required
- Not for use over open wounds and infected areas
- May not be for people with circulatory problems

PRECAUTIONS

- This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- The splint should fit snugly and comfortably without interfering with circulation.
- After heating splinting material, always check its temperature before applying it to the patient.

CARE AND CLEANING

Polyform® Light stay

The stay will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.

Fabric splint

Remove stay and Gel Shell pad and fasten straps. Hand wash fabric in warm water and mild soap. Rinse thoroughly. Do not wring. Dry at room temperature. Be sure the splint is completely dry before reapplying it.

Gel Shell pad

Hand wash in warm water and mild soap (hot water may yellow the pad). Rinse thoroughly. Press gently to remove excess water and let air dry. Be sure pad is completely dry before reapplying it.

INSTRUCTIONS FOR USE

The Rolyan Gel Shell Thumb Spica Splint consists of 3 parts:

- fabric splint with pocket for stay
- optional stay of $\frac{1}{16}$ (1.6mm) Polyform Light splinting material
- optional Gel Shell pad with hook attachment on back

The splint can be used with the stay only, with the gel only, or with both gel and stay.

Optional stay

1. Position the indented part of the stay between the MCP and CMC joints. The stay should fit along the radial aspect of the thumb, wrist and partial forearm (**A**). The distal end should

D

lie proximal to the IP joint. If necessary, the stay can be molded for a more exact fit by heating it for 30 seconds in water heated to 150° to 160°F (65–70°C).

2. Place the stay in the pocket in the splint.

Optional Gel Shell pad

3. Fasten the back of the pad to the surface of the pocket for the stay (**B** and **C**).
4. Remove the protective plastic sheet from the surface of the gel pad.

The splint

5. Position the splint and fasten the straps as shown in **D**,

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Patterson Medical Ltd.