Rolyan® Hot and Cold Discrimination Kit:

A629-1 and A629-003

INDICATIONS

- For assessing temperature discrimination

PRECAUTIONS

- **Always check temperature of probe before applying it to the patient.**

INSTRUCTIONS FOR USE

The kit includes the following components:

- two double-insulated thermal cups
- four aluminum temperature probes with dial thermometers on top
- one dial thermometer without a probe

1. All of the thermometers should record the same temperature. If any of them needs

adjusting, turn the head of the dial clockwise to lower the temperature reading and counterclockwise to increase it.

2. As a general guideline, fill the thermal cups with cold and hot tap water and ice cubes as necessary until one reaches a cold temperature of 40°F or 4°C and the other reaches a hot temperature of 110°F or 43°C. The thermometer without a probe can be used to determine water temperature quickly and to monitor it.
3. Remove the stoppers from the holes in the lids and put the lids onto the cups. Insert two probes into each cup. When the probes

register the water temperature, they are ready to use.

4. Remove one probe from each thermal cup and wipe dry.
5. Touch patient alternately with hot and cold probes to test temperature discrimination.
6. When the temperature registered by the cold probe rises or the hot probe falls, the probes can be returned to the thermal cups and the second set used.

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Patterson Medical, Ltd.