## Rolyan® Preformed Foot Drop Splint:

A634-1 through 3

## Rolyan® Foot Drop Splint with Expanded Heel:

A634-400 through 403

### INDICATIONS

- To help prevent or correct plantar flexion contractures
- To help prevent foot eversion and inversion

### CONTRAINDICATIONS

- Not for fixed bony contractures
- Not for use over open wounds and infected areas

### PRECAUTIONS

- **These splints are not to be used during ambulation.**
- **These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, discomfort, or skin discoloration or breakdown occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

### CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

### INSTRUCTIONS FOR USE

The splints include the following components:

- preformed splint of 1/8" (3.2mm) **Polyform®** splinting material lined with 1/4" (6.4mm) **Polycushion Padding®**
  - one long foam strap
  - two shorter foam straps with self-adhesive hook tabs
  - three hook coin pieces
1. To trim or make minor modifications to the splint, spot heat it with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C) until pliable. **Heating the entire splint may cause it to lose its preformed shape.**
  2. Apply the long foam strap so it crosses over the ankle as follows:

A

- Attach one hook coin piece just distal to the heel in the center of the foot portion.
  - Attach one hook coin piece on the medial side and the lateral side slightly proximal to the malleolus.
  - Attach the strap in a figure-of-8 manner, attaching it to all three hook coin pieces (a properly positioned strap is shown in **A**).
3. Attach the two shorter foam straps at the distal and proximal areas of the splint.

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**Patterson Medical Ltd.**