## Rolyan® Resting Pan Mitt Splint:

A762-SFX, MFX, LFX, SE, ME, LE, 231, 331, 431, 5504-61 through 5504-63, 5664-00 through 5664-02

*Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints listed.*

*Figures referred to in the instructions are on page 6.*

### INDICATIONS

- To help prevent deformity—possible indications include arthritis, hypertonicity, peripheral nerve injuries, and acute synovitis

### CONTRAINDICATIONS

- Not for the acute phase of most wrist or hand fractures

### PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

### CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water and allowed to dry thoroughly before reapplication.

### PROCEDURE

The splint blank can be applied to either hand—simply invert the splint blank.

1. Heat splint blank as follows:  
Soften in water heated to approximately 160°F (70°C) until pliable. (Refer to the instructions for Rolyan splinting materials for information about working with specific splinting materials.) Remove from heat pan and place on a towel on a flat surface. Blot excess water.

**Fabricating a volar splint** (see instructions 7–14 for a dorsal splint)

2. Flare the web space by stretching the outer ridge of the splinting material so it contours to the web space (A).
3. If possible, place patient's hand in supination to achieve a gravity-assisted position.
4. Mold the splint blank on the hand and forearm as shown in B.
5. The splint can be trimmed and the edges flared if necessary.

6. Apply appropriate straps. A completed volarly applied splint is shown in C.

### Fabricating a dorsal splint

7. Place hand on splint blank and mark the width of the hand at the MCPs and the thumb at the MCP (D).
8. Cut slits as marked.
9. Roll back splinting material at the slits so the folds will be to the outside of the completed splint (the proximal fold should face up and the distal fold down) (E). Roll back just enough material so the digits fit through the slits.
10. Flare the web space by stretching the outer ridge of the splinting material so it contours to the web space (A).
11. If possible, place patient's hand in pronation to achieve a gravity-assisted position.
12. Mold the splint blank on the hand as shown in F. Roll back the dorsum of the splint to clear the MCPs. Mold the splint to the forearm.
13. The splint can be trimmed and the edges flared if necessary.
14. Apply appropriate straps. A completed dorsally applied splint is shown in G.

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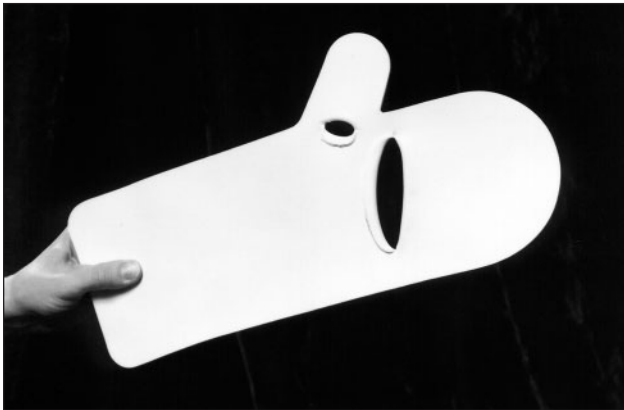
PATTERSON  
MEDICAL  
**SAMMONS  
PRESTON**  
Sammons Preston  
A Patterson Medical Company  
W68 N158 Evergreen Blvd.  
Cedarburg, WI 53012

homecraft  
Rolyan  
A Patterson Medical Company

Authorized European Representative  
**Homecraft Rolyan**  
A Patterson Medical Company  
Nunn Brook Road, Huthwaite  
Sutton-In-Ashfield, Nottinghamshire NG17 2HU  
United Kingdom  
Phone: 08702 423 305  
Fax: 08701 660 211

**International**  
Phone: +44 1623 759 146  
Fax: +44 1623 726 886

G

© February 1993, Revised 10/93, 11/95, 7/97, 8/98, 11/04, 3/05, 2/09  
Patterson Medical Products, Inc.  
9104.455C