Rolyan® Burn Splint:

A765-1 through 6

INDICATIONS

- To position the hand following burns, tendon transfers, and trauma

PRECAUTIONS

- This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**

- After heating splinting material, always check its temperature before applying it to the patient.**

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water. It should be allowed to dry thoroughly before reapplication.

PROCEDURE

The Rolyan Burn Splint positions the extremity approximately as follows:

- wrist in 20° to 30° of extension

- MCPs in 60° to 70° of flexion
 - IPs in full extension
 - thumb in 45° of extension midway between radial and palmar abduction
- Place bandaged hand in splint to evaluate fit.
 - If necessary, to trim or modify the splint, which is made from 1/8" (3.2mm) **Ezeform®** splinting material, spot-heat it with a heat gun or in a heat pan of water at a temperature of 160° to 170°F (70°–75°C) for one minute or until pliable. **Heating the entire splint may cause it to lose its preformed shape.**
 - Apply straps as appropriate to hold splint in place.

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Patterson Medical, Ltd.