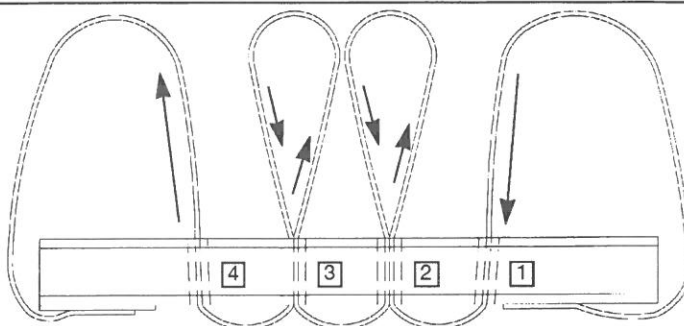
Rolyan* Palm Protector with Finger Separators:

A812-01 and A812-02

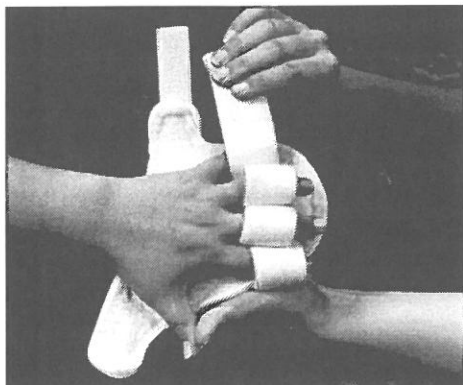
Strap fastens to hook
Band wordt bevestigd aan haakband
Bande fixée à la partie agrippante
Das Band wird am Haken befestigt
La fascetta si fissa
La tira se fija a la lengüeta

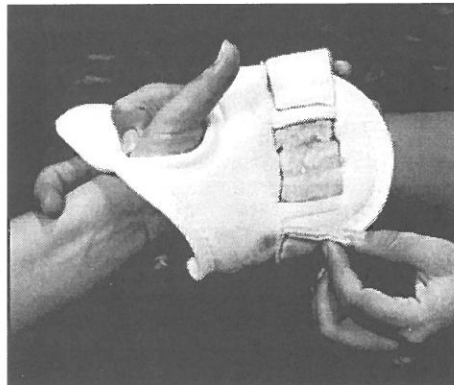
Strap sewn to splint
Band aan spalk genaaid
Bande installée sur l'attelle
An der Schiene angenähtes Band
Fascetta cucita allo splint
Tira cosida a la férula

A

B

C

D

INDICATIONS

- May be used for moderate to severe finger spasticity or contractures

CONTRAINDICATIONS

- Not for use if uncontrolled edema or open sores are present

PRECAUTIONS

- **The splint should fit snugly and comfortably without interfering with circulation.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be**

discontinued and a healthcare professional consulted.

CARE AND CLEANING

- Fasten straps. Hand wash fabric in warm water and mild soap. Do not wring. Dry at room temperature. Be sure the splint is completely dry before reapplying it.
- Unthreading the finger separating strap from the splint before washing will reduce drying time. To rethread the strap, see A.

INSTRUCTIONS FOR USE

1. With the straps open, place the fingers into the finger separators (B).
 2. Place the thumb through the thumb hole (C).
 3. Attach the finger strap around the palmar surface of the fifth digit (C).
 4. Attach strap on dorsum of wrist.
 5. Adjust tension of straps for a proper fit.
 6. A properly applied splint is shown in D.
- (Design copyright Smith & Nephew.)

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* Trademark of Smith & Nephew
Smith & Nephew, Inc.
Germantown, WI 53022-8205 U.S.A.
Smith & Nephew Homecraft Ltd.
Rehabilitation Division, Smith & Nephew, Inc.