Rolyan® Light Hand-Based Thumb Spica Splint:


A907-11 through A907-18

Rolyan® Light Forearm-Based Thumb Spica Splint:

A907-1 through A907-8

A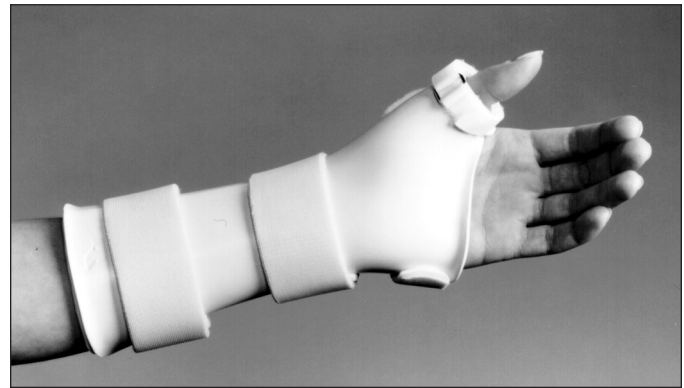

B

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to both of the splints.

INDICATIONS

- To support the CMC and immobilize the MCP joints of the thumb
- May be used for arthritis, skier's thumb, and gamekeeper's thumb

Rolyan Light Forearm-Based Thumb Spica Splint

- For additional immobilization of the wrist
- May also be used for deQuervain's syndrome

CONTRAINDICATIONS

- Not in the presence of severe edema
- Not for the acute phase of most wrist or thumb fractures
- Not over open wounds and infected areas

PRECAUTIONS

- **These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**

- **Be careful that the splint is not so tight that circulation is impaired.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

INSTRUCTIONS FOR USE

The splints include the following components:

- splint of 1/16" (1.6mm) **Polyform®** Light material
- Plastazote® liner
- 1/2"-wide (1.3cm) self-adhesive D-ring strap
- 1"-wide (2.5cm) self-adhesive foam strap
- self-adhesive hook coin

Forearm-Based Splint

- two 1 1/2"-wide (3.8cm) foam straps with self-adhesive hook tabs

Making modifications

1. Before permanently attaching liner to splint, apply the splint (with liner in place) to check for fit. To trim or modify the splint, remove it

and spot-heat it with a heat gun or in a heat pan of water at a temperature of 150° to 160° F (65°–70°C) for 30 seconds or until pliable. **Heating the entire splint may cause it to lose its preformed shape.** Polyform Light has a high degree of drape—for better results when modifying the material, shape it with stroking movements.

2. Remove the backing from the adhesive strips on the back of the liner and apply liner to splint.

Applying straps

Forearm-Based Splint

3. With the splint on the patient, attach the two 1 1/2"-wide (3.8cm) foam straps around the forearm.

Both splints

4. With the splint on the patient, position the 1"-wide (2.5cm) strap across the dorsum of the hand and attach the self-adhesive loop on the strap to the ulnar side of the splint. Attach the self-adhesive hook coin to the radial side of the splint and fasten the strap.
5. Position the D-ring strap around the thumb area so the strap pulls in the direction of the thumb overlap before passing through the D-ring. Attach the self-adhesive hook tab to the splint.
6. A properly applied Hand-Based Thumb Spica Splint is shown in **A**; a Forearm-Based Thumb Spica Splint is shown in **B**.

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© 1991, Revised 4/97, 8/98, 11/03, 4/05, 3/07