Rev. 0 - 05/11_108955

Material

Polyamid (PA)
Polyurethan (PUR)
Aluminium AL Mg3 W19
Polyethylen (PE)
Polyoxzmethylen (POM)
Polyester (PES)

Material

Polyamide (PA)
Polyurethane (PUR)
Aluminium AL Mg3 W19
Polyethylene (PE)
Polyoxzmethylene (POM)
Polyester (PES)

Bauerfeind AG
 Weissendorfer Strasse 5
 07937 Zeulenroda-Triebes
 Germany
 Phone +49 (0) 36628-66-40 00
 Fax +49 (0) 36628-66-44 99
 E-mail info@bauerfeind.com

RhizoLoc

Stabilorthese
 Stabilizing orthosis
 Orthèse de stabilisation
 Ortesis estabilizadora

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EN Dear customer,

Thank you for placing your trust in a Bauerfeind product. The RhizoLoc® you have purchased is a high-quality medical product.

Please read through all of these instructions for use carefully, because they contain important information about the use, indications, contraindications, application, care and disposal of the RhizoLoc®. Please keep these instructions safe, as you may want to read them again later. If you have any questions, please contact your physician or medical retailer.

Intended use

The RhizoLoc® is an anatomically contoured stabilizing orthosis for the thumb. It stabilizes and, if necessary, immobilizes the thumb carpometacarpal joint and thumb basal joint. This is made possible by aluminum frames. A thumb tab can be used to vary the extent of limitation of the thumb basal joint. With its stabilizing effect, the RhizoLoc® helps prevent undesired movements and relieve pain.

Indications

- collateral ligament injuries of the thumb basal joint (e.g. skier's thumb)
- irritation in the region of the thumb carpometacarpal joint and thumb basal joint (e.g. osteoarthritis)

Risks of using this product

Caution*

RhizoLoc® must only be worn in accordance with the specifications contained in these instructions and used for the areas of application (places of use) listed.

* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

To guarantee the best possible fit of the RhizoLoc®, it is essential to check whether the product is adequately shaped and whether it needs to be anatomically reshaped by a specialist¹. Only a trained specialist may carry out the (initial) fitting of the orthosis and provide instruction.

The sewn-in label of the RhizoLoc®, which contains information on the product name, size, manufacturer, washing instructions and CE certification, is located on Rhizobic outside of the wrist strap.

Use in combination with other products, e. g. as part of compression treatment (arm stockings), must be discussed beforehand with your physician.

Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay. No product liability is accepted in the event of improper use. Avoid the use of ointments, lotions or any substances including grease or acids. Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed.

Do not put the RhizoLoc® on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps of the RhizoLoc® and, if necessary, check the size of your RhizoLoc®.

Contraindications

Hypersensitivity reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and worn after consultation with your physician:

- skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- impaired sensation and circulatory disorders of the arm/hand (e.g. diabetes mellitus)
- impaired lymph drainage, including soft tissue swellings of uncertain origin located away from the fitted support

Application instructions

Putting on the RhizoLoc®

A - wrist strap, B - thumb strap,

C - thumb tab

- 1 Open the two Velcro straps on the RhizoLoc®. Loosen the straps without pulling them out of the guide.
- 2 Put the RhizoLoc® on the affected thumb. To do this, guide your hand through the bottom wrist strap and your thumb through the top thumb strap.
- 3 Close the wrist strap. Then close the thumb strap by closing the Velcro at the side next to the thumb tab.
- 4 Following consultation with your physician, you can use the thumb tab to adjust the level of stabilization of the thumb basal joint depending on the phase of your treatment.

Application instructions

Taking off the RhizoLoc®

Open the thumb and wrist straps. Guide your hand downward out of the RhizoLoc®.

Cleaning instructions

Note: Never expose the RhizoLoc® to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may damage the material, which can impair the effectiveness of the RhizoLoc®.

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

Fasten all Velcro straps before washing to avoid damage. Wash the RhizoLoc® separately by hand, using a mild detergent.

Part of the body this product is used for

In accordance with the indications (thumb).
See intended use.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The RhizoLoc® is supplied in an anatomically contoured state. If necessary, the RhizoLoc® can be contoured further. The orthosis is available in right- and left-handed designs.

Technical specifications / parameters, accessories

The RhizoLoc® is a stabilizing orthosis for the thumb. There are no spare parts available.

Notes on reuse

The RhizoLoc® is intended for your treatment only. It has been adapted specifically for you. For this reason, do not pass the RhizoLoc® on to other people.

Warranty

Warranty is provided in accordance with the statutory provisions as well as the terms of supply and contractual terms of Bauerfeind AG (AGB), as long as all the points mentioned in these instructions are observed.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: February 2011

Specialists¹

General notes

- Check the optimal and individual fit of the RhizoLoc® after the first fitting. Make corrections if necessary.
- Practice the correct procedure for putting on the orthosis with the patient.
- The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.
- The RhizoLoc® is intended for the treatment of one patient only.

The aluminum stays are anatomically contoured, but they may still need to be individually adjusted to suit the patient's anatomy. To do this, apply a little force to contour the aluminum stays into the anatomically correct form.

The thumb tab to limit the thumb basal joint must be firmly strapped in and under tension when treatment begins and must contain no folds. It is the physician's responsibility to adjust the level of restriction of movement of the thumb.

If you have any complaints, please contact your medical retailer only. We would point out that only clean goods can be dealt with.

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