Rev. 2 - 04/11_103894

Bauerfeind AG
 Weissendorfer Strasse 5
 07937 Zeulenroda-Triebes
 Germany
 Phone +49 (0) 36628-66-40 00
 Fax +49 (0) 36628-66-44 99
 E-mail info@bauerfeind.com

Motion is Life: www.bauerfeind.com

AchilloTrain® Pro

Aktivbandage
 Active support
 Orthèse active
 Vendaje activo

Motion is Life: www.bauerfeind.com

Material

Silikon*

Polyamid (PA)

Elastodien (ED)

Baumwolle (CO)

Viskose (CV)

Elastan (EL)

Polyurethan (PU)

Polyester (PES)

Polyacrylat

Druckfolie

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EN Dear client,

thank you for choosing a product from Bauerfeind. AchilloTrain® Pro is a high-quality medical product.

Please read through these instructions for use carefully. If you have any questions please contact your doctor, your local retailer or our Technical Service department.

AchilloTrain® Pro, the active Achilles tendon support is anatomically knitted which ensures an excellent fit. The massage effect of the shaped viscoelastic insert around the Achilles tendon area promotes lymph drainage and circulation and helps reduce swelling and bruising. The stretchable knitted section around the shin also facilitates donning of the support.

⚠ AchilloTrain® Pro must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.

Indications

- Painful irritation of the Achilles tendon (achillobdunia) and adjacent tissues
- After surgery, e.g. tears in Achilles tendon

Side effects

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses¹ applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and worn after consultation with your doctor:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the foot, e.g. Diabetes mellitus.
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Fitting AchilloTrain® Pro

1. Grasp the top edge of the support and pull it over the foot until the heel is located in the heel-wedge section (Figures 1 and 2).
2. Ensure that the Achilles tendon is securely held by the shaped insert.
3. Adjust the fit of the support as necessary.

⚠ Important Information

- AchilloTrain® Pro is a prescribed product that should be worn under a physician's guidance. AchilloTrain® Pro should only be worn in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product is intended to be fitted only once and to one patient.
- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your doctor without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your doctor.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.
- The product can be disposed of without any restrictions.

Washing instructions and product care

⚠ All Train products² can be machine washed separately at 30 °C with liquid detergent. Please note the following when washing your Train products:

1. For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.
2. If possible, also remove pads/inserts and their terrycloth covers before washing.

We recommend using a mesh bag. Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric.

Please wash your Train product regularly so that the compression of the knitted fabric can be retained.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² Standardized washing tests show that after 50 washing machine cycles full functionality of the Train® active support is retained. (Liquid detergent was used.)

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