

TheraFit Premium Soft Goods

Shoulder Harness - Vest - Chest Strap - 4 Point Hip Belt - Hip Belt - Rear Pull Hip Belt - Center Pull Hip Belt



Custom Seating 2-25

TheraFit Soft Goods

Trays 77-98

Tray Attaching 99-103

Sip-N-Puff / Joystick 104-107

Patient Transfer 108-111

Therapy 112-115

Independent Living 116-121

TheraFit is our new line of High Quality Soft Good Supports.

These supports are redesigned products that use the best materials and finishing techniques to provide superior positioning for your clients.

As you've come to expect from TheraFit, they are competitively priced to address your reimbursement challenges.



- Shoulder Harness page 37
- Stretch Vests page 38
- Vest Sizing Guide page 39
- Chest Straps page 40
- 4 Point Hip Belts page 41
- Single Pull Hip Belts page 42
- Rear Pull Hip Belts page 43
- Center Pull Hip Belts page 44



TheraFit Shoulder Harness

- Heavy duty non-stretch cushioned harness with reinforced webbing
- Stand behind user and pull finger loops to adjust top straps

Padded Area Dimensions

| | |
|---------------------------------|--------------------------|
| SMALL13/4" x 12" | MEDIUM . . .2" x 13 1/2" |
| LARGE2 1/2" x 15 1/2" | X-LARGE . . .23/4" x 18" |

With Metal Clips


| |
|----------------------|
| SMALL61401 |
| MEDIUM . . .61501 |
| LARGE61701 |
| X-LARGE . . .61801 |

With Cam Locks

| |
|----------------------|
| SMALL61402 |
| MEDIUM . . .61502 |
| LARGE61702 |
| X-LARGE . . .61802 |

With Metal Clips and Cam Locks

| | |
|----------------------|--------------------|
| SMALL61403 | MEDIUM . . .61503 |
| LARGE61703 | X-LARGE . . .61803 |

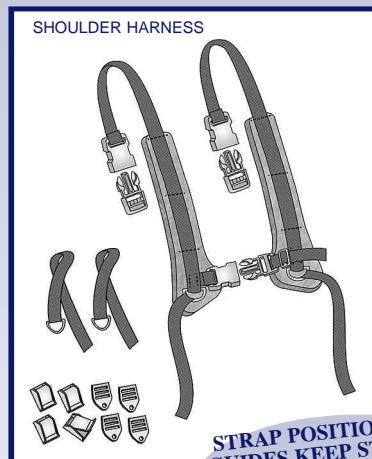


*A harness is not a safety device.
It is to be used only for positioning.*

H HARNESS X HARNESS SHOULDER HARNESS

PARTS DESCRIPTION

- ◆ All parts shown here are not included with every harness
- ◆ Parts included vary depending on which harness you have

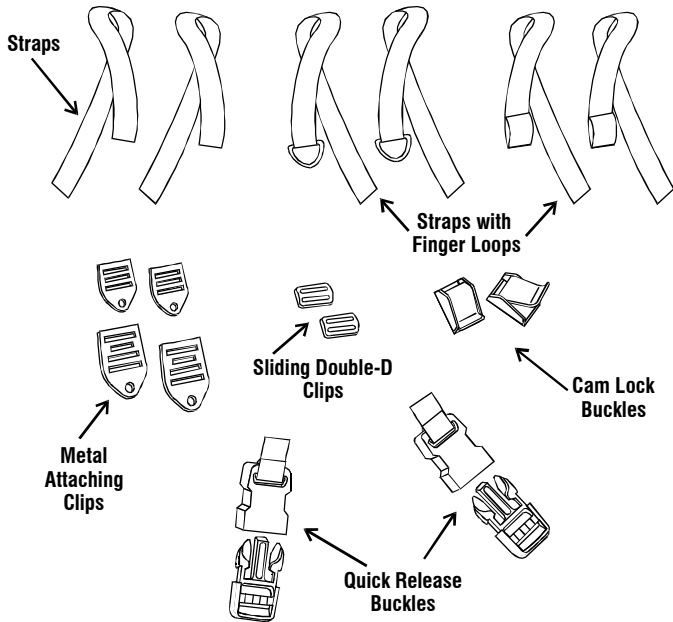


This chest harness is a **Kit** that you assemble as needed for your particular application. This sheet illustrates **examples** of the many ways to use the various straps, clips, and buckles.

HOW TO ATTACH A QUICK RELEASE BUCKLE TO A STRAP

- ◆ Pull **Strap** for desired tight fit.



When threading a Strap, the Buckle piece must have the correct side up. The part the Strap goes around has a flat side that belongs on top, and a triangular side that belongs on the bottom.

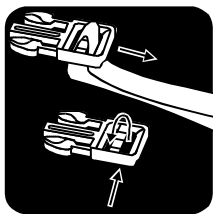


Thread the Strap as shown. If the Strap has a Finger Loop, the Finger Loop should be on the top.

Always pull the top Strap to tighten.


HOW TO ATTACH A STRAP TO THE CHAIR

- ◆ Secure **Cam Lock Buckle** and/or **Metal Attaching Clip** to chair, then depending on the width of the strap, thread strap as shown.


Cam Lock Buckle

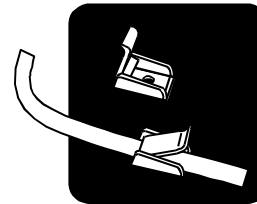


Metal Attaching Clip & .75", 1", Or 1.5" Strap

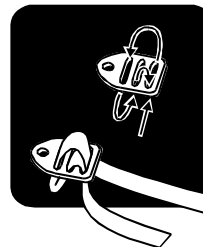

Metal Attaching Clip & 2" Strap

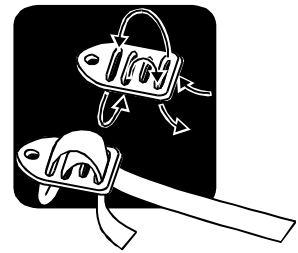

- ◆ Thread the strap through a **Sliding Double-D Clip** as shown, looping strap around the wheelchair tube, and pull tight to hold harness in desired position.

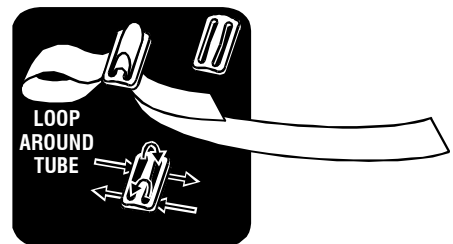

REMOVING USER FROM CHAIR

- ◆ Use **Quick Release Buckles** so you don't have to refit all the adjustments the next time user gets into chair.

IMPORTANT NOTICE

- ◆ **THIS IS NOT A SAFETY DEVICE. IT IS TO BE USED ONLY FOR POSITIONING. IT MUST BE USED IN CONJUNCTION WITH A METAL SEAT BELT!**