


Gel-Foam Pad

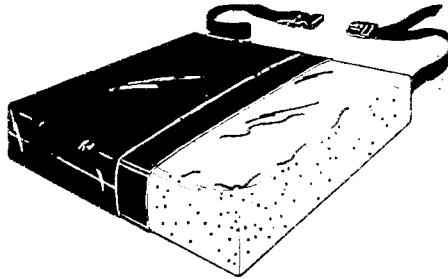

Reorder #	Description	Size
☐751010	Gel-Foam Pad	16" x 16" x 2 ½"
☐751011	Gel-Foam Pad	18" x 16" x 2 ½"
☐751012	Gel-Foam Pad	20" x 16" x 2 ½"
☐751040	Thin-Line Gel-Foam Pad	18" x 16" x 1 ½"
☐751045	Thin-Line Gel-Foam Pad w/Firm Foundation	18" x 16" x 2"

Effective Design

This two-chamber wheelchair cushion combines the comfort and therapeutic effectiveness of gel and foam. The high-density foam base prevents bottoming out, provides stability, and increases comfort. The waterbased gel surface cools the skin to prevent perspiration while relieving interface pressure.

New Low-Shear Cover

The outer cover is made of two-way stretch Low-Shear fabric. It moves with the wheel-chair user to reduce skin-damaging shear and friction. The Low-Shear cover is completely launderable. It may be washed and dried at temperatures that do not exceed 180°F (82°C). Do not use bleach.

Important Safety Feature

The Low-Shear cover has a safety strap that prevents the cushion from slipping off the seat. This strap must be secured to the wheelchair frame.

Easy Care

The Low-Shear cover is launderable (see above). The incontinent-proof vinyl inner cover is factory sealed to prevent urine from soaking through to the foam. Spray-and-wipe cleanser and mild disinfectant may be used to clean the vinyl.