

Double Bolster
Item #081545045



Use:

The Double Bolster with AEGIS Microbe Shield® allows easy bed entry and exit for the patient while also providing protection from falling out of bed. It attaches easily to the bed frame with quick-release buckles and allows patients maximum sleeping surface. It also acts as a pressure-relieving positioning aid to increase comfort. The AEGIS Microbe Shield® antimicrobial technology controls microorganisms that cause odors and stains, and it slows the degradation of products. It will not wash out of the product. Lasts during the life of the product. Effective against virtually all problem microbes including bacteria, fungi and algae. It does not rub off or migrate onto skin.

Dimensions: 25" L x 49" W (Inner width between pillows: 23", Straps: 33" L x 4" W)

Instructions:

- Position the bolster in the desired position on the bed
- Secure the bolster to the bed using the straps provided
- Position patient in bed on bolster
- Reposition the bolster/patient as necessary

Latex Free

**Fire Retardant
By CA-117 Standards**

Cleaning Instructions:

Wipe outer cover with mild soap and/or disinfectant.



Hand wash only



Do Not Bleach

