

#72124 “No-Lift” Turner

Instructions for Applications, Use & Care

“No-Lift” Turner allows caregivers to turn the patient in bed for proper positioning. Helps reduce the risk of injury to staff.

Use

1. Lower head of bed so mattress is completely flat.
2. Roll patient onto side and position Turner under patient using draw sheet method.
3. Position Turner under patient so that yellow nylon webbing is perpendicular to the patient. *Figure 1*

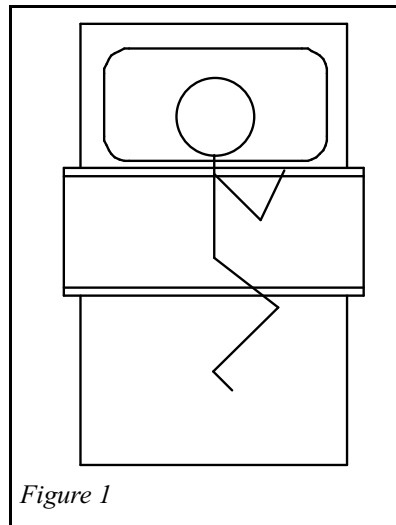


Figure 1

4. Roll patient onto their back and adjust Turner so the patient’s buttocks and shoulders are on the Turner sheet. *Figure 2*
5. Grab hold of top layer of Turner on opposite side you wish to roll patient.
6. Pull toward you in a slight upward motion. This will rotate the patient onto their side.

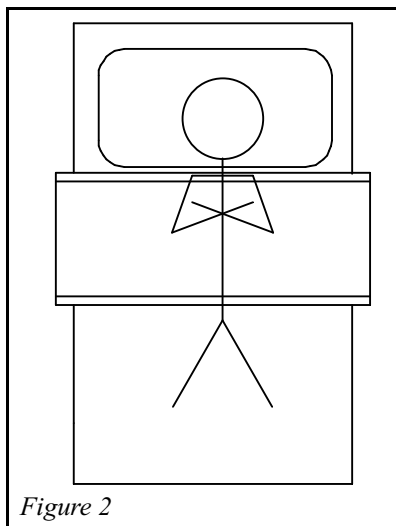


Figure 2

7. Position pillows between patient’s knees for comfort and use positioning wedge behind patient to maintain side-lying position.
8. “No-Lift” Turner may remain under patient between positioning changes as long as side rails are utilized.
9. To roll patient to back to opposite side, grab top layer of “No-Lift” Turner on opposite side you wish to roll patient. Example: If you wish patient to ultimately be lying on their right side, pull on left side of Turner.
10. Repeat step 7 to maintain positioning.
11. Remove “No-Lift” Turner as would a typical draw sheet.
12. “No-Lift” Turner must be removed prior to sitting patient at edge of bed or attempting to transfer patient from bed.

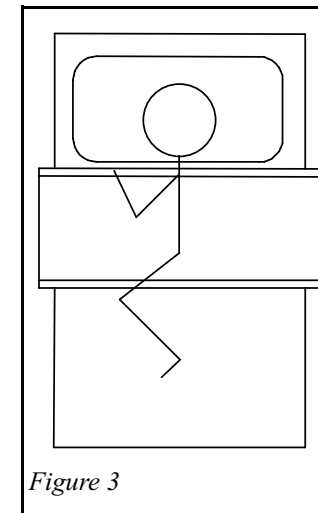


Figure 3

Precautions

- Never place product on floor. Product is extremely slippery.
- “No-Lift” Turner is designed to slide the patient, NOT to lift them in any way.
- DO NOT use near edge of bed.
- DO NOT leave under unsupervised patient.
- Regularly check “No-Lift” Turner for signs of wear. DO NOT use if there is any sign of deterioration.
- Keep instruction sheet with product for reference and training.
- “No-Lift” Turner is intended to reduce the risk of injuries to staff. It cannot replace proper body mechanics, which are recommended at all times.

Care

- **If heavily soiled:** Machine wash with mild detergent. Air dry.
- **Between patient use:** Use spray disinfectant. Towel dry.

To Reorder

#72124 “No-Lift” Turner, 58” W x 29” L

Manufactured by:

 **AliMed[®] inc.**
Dedham, MA 02026-9135

5523 R1019

©2001 AliMed inc. All rights reserved.

#75638 “No-Lift” Turner

Instructions for Applications, Use & Care

“No-Lift” Turner allows caregivers to turn the patient in bed for proper positioning. Helps reduce the risk of injury to staff.

Use

1. Lower head of bed so mattress is completely flat.
2. Roll patient onto side and position Turner under patient using draw sheet method.
3. Position Turner under patient so that yellow nylon webbing is perpendicular to the patient. *Figure 1*


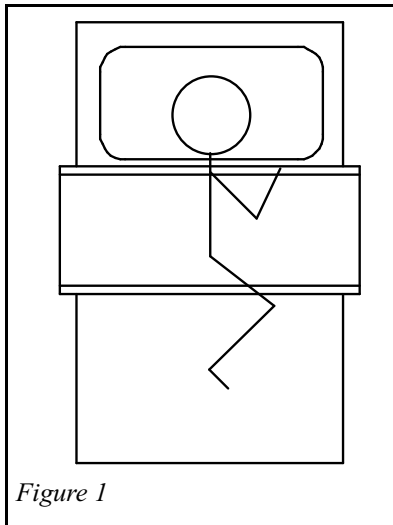


Figure 1

4. Roll patient onto their back and adjust Turner so the patient's buttocks and shoulders are on the Turner sheet. *Figure 2*
5. Grab hold of top layer of Turner on opposite side you wish to roll patient.
6. Pull toward you in a slight upward motion. This will rotate the patient onto their side.

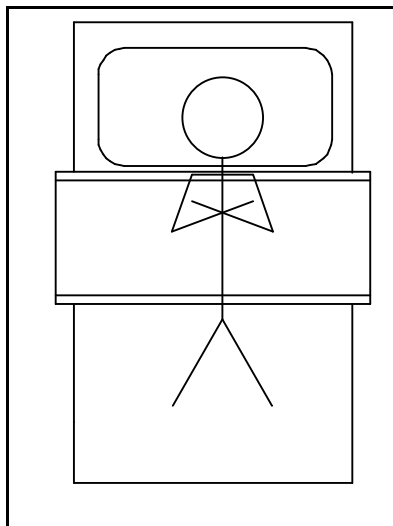

7. Position pillows between patient's knees for comfort and use positioning wedge behind patient to maintain side-lying position.
8. “No-Lift” Turner may remain under patient between positioning changes as long as side rails are utilized.
9. To roll patient to back to opposite side, grab top layer of “No-Lift” Turner on opposite side you wish to roll patient. Example: If you wish patient to ultimately be lying on their right side, pull on left side of Turner.
10. Repeat step 7 to maintain positioning.
11. Remove “No-Lift” Turner as would a typical draw sheet.
12. “No-Lift” Turner must be removed prior to sitting patient at edge of bed or attempting to transfer patient from bed.

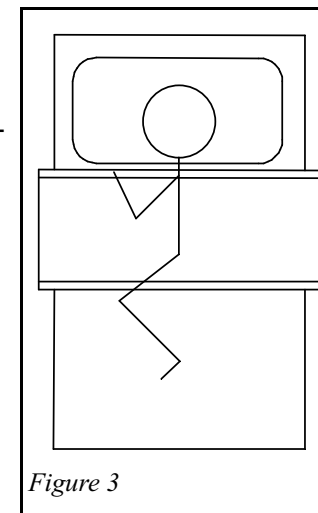


Figure 3

Precautions

- Never place product on floor. Product is extremely slippery.
- “No-Lift” Turner is designed to slide the patient, NOT to lift them in any way.
- DO NOT use near edge of bed.
- DO NOT leave under unsupervised patient.
- Regularly check “No-Lift” Turner for signs of wear. DO NOT use if there is any sign of deterioration.
- Keep instruction sheet with product for reference and training.
- “No-Lift” Turner is intended to reduce the risk of injuries to staff. It cannot replace proper body mechanics, which are recommended at all times.

Care

- **If heavily soiled:** Machine wash with mild detergent. Air dry.
- **Between patient use:** Use spray disinfectant. Towel dry.

To Reorder

#75638 “No-Lift” Turner, 58” W x 29” L

Manufactured by:



R1019

©2001 AliMed inc. All rights reserved.