

# A dynamic solution for a dynamic problem<sup>®</sup>

What is the *SaeboStretch*?

How is the *SaeboStretch* different from current splints?

The *SaeboStretch*<sup>®</sup> is a revolutionary new, award-winning dynamic hand splint for individuals suffering from neurological injuries such as stroke. The splint helps to prevent joint damage while improving and maintaining range of motion.



## Positions the Hand:

The *SaeboStretch* provides maximum positioning utilizing the following:

- Proprietary new dynamic hand piece that comes in three varying grades of resistance to better match your patient's specific needs
- Proprietary new strapping system that utilizes key anatomical points of control to maximize control and ensure an intimate fit
- Malleable wrist and thumb mounts allow the therapist to further customize the fit
- New non-slip strap material and design help facilitate keeping the fingers in the desired position
- Proprietary Breath-O-Preen covering that allows moisture to wick away, enhancing skin integrity and hygiene
- Special padding to help maintain the integrity of the palmer arch



Other splint



*SaeboStretch*

Give your patient's hand the best possible chance for recovery

## Protects the Joints:

Traditional splints are made from hard/rigid material that do not allow for any movement of the fingers. The SaeboStretch has a unique dynamic hard piece that allows the fingers to move through flexion caused by postural changes, associated reactions and increased tone. The same stretch technology then gradually repositions the fingers into extension. The SaeboStretch includes three different hand pieces that offer various grades of resistance so you can further match your patient's specific needs.



"The SaeboStretch finally allows the control of my patient's hand that I have always wanted. He stood, he walked, and even yawned without losing position of his fingers. Thanks for making the product all therapists have desired."


**Liz Miner**  
Occupational Therapist

"It really is the most comfortable hand splint I've ever worn. Had the SaeboStretch been available when I had my stroke, I think I might have more function and use of my hand today."

**John Whitfield**  
Stroke Survivor

Indications: SaeboStretch is appropriate for patients with minimum to moderate tone and soft tissue shortening. It is NOT appropriate for patients with severe tone and contractures.

Sizing instructions: measure across the proximal phalanges of digits 2-5 just proximal to the PIP joints. Make sure the fingers are in the adducted position. Do not wrap the tape down the lateral or medial side of the hand.

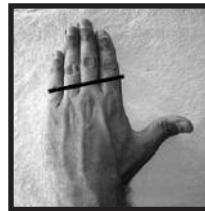

Consider the length of the fingers when deciding between 2 different sizes. (i.e. shorter fingers = smaller size)

### SaeboStretch

Left	Right	Width of 2-5th digits just proximal to the PIP joints	Size
0815-55-093	0815-55-127	2¼" to 2¾"	Small
0815-55-101	0815-55-135	2¾" to 3¼"	Medium
0815-55-119	0815-55-143	3¼" to 4"	Large

### Replacement Liners

Left	Right	Size
0815-55-184	0815-55-150	Small
0815-55-192	0815-55-168	Medium
0815-55-200	0815-55-176	Large

### Replacement Strap Set

0815-55-218	Small
0815-55-226	Medium/Large

\*Note product must be fitted by a therapist or orthotist.

When choosing a splint, request the best. Ask for the SaeboStretch. To learn more about this revolutionary new splint, please call 1-800-323-5547 or contact your local Patterson Medical Representative

## What happens when you put a dynamic hand in a static splint?

- Joint Deformities
- Contractures
- Hypermobility



www.pattersonmedical.com

To order call:  
**1-800-323-5547**  
In Canada call 1-800-665-9200

Fax toll free:  
**1-800-547-4333**  
In Canada fax 1-877-566-8611