Instructions below are for International TheraPearl Sports packs only. Please see individual product for heating instructions.

English-13576-ART

THERA°PEARL® Sports Pack

DIRECTIONS:

THERA°PEARL pack should be applied for 20 minutes, or as doctor recommended.

Remove and wait for at least 20 minutes, Repeat as needed.

COLD THERAPY~Freeze for at least 2 hours.

HEAT THERAPY~Microwave for time indicated below and test pack temperature.

Heat for additional 5 seconds as needed until desired temperature is reached.

Suggested heating time for (1) one THERA°PEARL Sports Pack		
700 Watts ~ 40 Seconds	1000 Watts ~ 35 Seconds	1250 Watts ~ 30 Seconds

CAUTION: Always test pack temperature prior to application. Prolonged application may cause injury, including burns. Do not use on infants. Do not use if you have poor circulation. Do not use heat therapy on stitches or an open wound. Do not fall asleep using pack. Use only under adult supervision. For extreme, prolonged, or chronic pain, contact a physician. Use only as directed. Heating beyond suggested time may damage or destroy THERA°PEARL pack. Do not puncture pack and/or ingest contents. For external use only.

See website for additional cautions & warnings.

www.therapearl.com

Patent Notice: This product and/or packaging may be protected by pending or issued U.S. or foreign intellectual property, including (but not limited to) patents, Industrial Design Registrations and/or Registered Community Designs. See www.therapearl.com/patent for specific details.

Our pearls are our trademark. Look for the distinctive, ornamental appearance of our pearls as a sign of our quality. Thera°Pearl®, A New Degree of Relief and Pearl Technology™ trademarks are property of Performance Health and/or its affiliates and may be registered in the United States and other countries. Unauthorized use is strictly prohibited. ©2020 TheraPearl, LLC. All rights reserved.

Ausmedic Australia Pty Ltd, Unit 3, 3 Basalt Rd Pemulwuy NSW 2145, 1300 473 422, performhealth.com.au

 The Hygienic Corporation
1245 Home Avenue
Akron, OH 44310, USA
+1-877-732-7509

EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague
The Netherlands

13576-R03 Made in China

Serious events experienced from using this product should be reported immediately to The Hygienic Corporation and to the local Competent Authority.

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HOW TO USE: COLD therapy ~ Freeze for at least 2 hours.
HEAT therapy ~ Heat in a microwave for the time and at the power indicated below. Packs are designed to retain a therapeutic temperature for approximately 20 minutes.

Suggested Heating Time for Thera°PEARL® Sports Pack		
700W ~ 40 seconds	1000W ~ 35 seconds	1250W ~ 30 seconds

Wait for at least 20 minutes before reapplication.
If necessary, reapply the pack for 20 minutes.

CAUTION: Always test pack temperature prior to application. Prolonged application may cause injury, including burns. Do not use on infants. Do not use if you have poor circulation. Do not use heat therapy on stitches or an open wound. Do not fall asleep using pack. Use only under adult supervision. For extreme, prolonged, or chronic pain, contact a physician. Use only as directed. Heating beyond suggested time may damage or destroy Thera°PEARL® pack. Do not puncture pack and/or ingest contents. For external use only. See website for additional precautions and warnings:
www.therapearl.com

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Thera°PEARL® Sports Pack with Adjustable Strap

EN DIRECTIONS: THERA°PEARL® pack should be applied for 20 minutes, or as doctor recommended. Remove and wait for at least 20 minutes. Repeat as needed. **COLD THERAPY:** Freeze for at least 2 hours. **HEAT THERAPY:** Microwave for time indicated below and test pack temperature. Heat for additional 5 sec. as needed until desired temperature is reached.

Suggested heating time for one THERA°PEARL® Sports Pack with Strap: 700 Watts: 35 seconds - 1000 Watts: 30 seconds - 1250 Watts: 25 seconds. **CAUTION: Always test pack temperature prior to application.** Prolonged application may cause injury, including burns. Do not use on infants. Do not use if you have poor circulation. Do not use heat therapy on stitches or an open wound. Do not fall asleep using pack. Use only under adult supervision. For extreme, prolonged, or chronic pain, contact a physician. Use only as directed. Heating beyond suggested time may damage or destroy THERA°PEARL pack. Do not puncture pack and/or ingest contents. For external use only. See website for additional cautions & warnings: www.therapearl.com

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THERAPEARL combines the healing power of HOT & COLD therapy into one convenient pack. Enjoy extraordinary relief from your everyday aches & pains and feel the optimal therapeutic benefit as our Pearl Technology® conforms to your body.

INTENDED USE: Reusable therapeutic hot & cold pack for use with common aches, pains, swelling and bruising.

COLD THERAPY ~ Freeze for at least 2 hours.

HEAT THERAPY ~ Heat in microwave for specified time. **DIRECTIONS:** THERAPEARL pack should be applied to skin for 20 minutes or as doctor recommended. Remove and wait at least 20 minutes. Reapply pack for 20 minutes, if needed. See full directions on product prior to use and application. Store in freezer or airtight container, out of direct light.

CAUTION: Always test pack temperature prior to application. Prolonged application may cause injury, including burns. Do not use on infants. Do not use if you have poor circulation. Do not use heat therapy on stitches or an open wound. Do not fall asleep using pack. Use only under adult supervision. For extreme, prolonged, or chronic pain, contact a physician. Use only as directed. Heating beyond suggested time may damage or destroy THERAPEARL pack. Do not puncture pack and/or ingest contents. For external use only.

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TheraPearl®, A New Degree of Relief® and Pearl Technology® trademarks are property of Performance Health and/or its subsidiaries and may be registered in the United States and other countries. Unauthorized use is strictly prohibited. ©2019 Performance Health. All rights reserved.

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Patent Notice: This product and/or packaging may be protected by pending or issued U.S. or foreign intellectual property, including (but not limited to) patents, Industrial Design Registrations, and/or Registered Community Designs. See www.therapearl.com/patent for specific details.

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Made in China • Fabriqué en Chine P10982 REV0

Serious events experienced from using this product should be reported immediately to The Hygenic Corporation and to the local Competent Authority.

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Our pearls are our trademark. Look for the distinctive, ornamental appearance of our pearls as a sign of our quality. Patent(s) and/or design registrations issued or pending. See www.therapearl.com patent for specific details. ©2020 TheraPearl, LLC. All rights reserved.